

# Bed Side Unit

Gama 500 Series Nurse Call System

NCS-501

The 501 Bed Head unit include all required component to communicate with the Nurse Station 500 Series. This bed head unit also have remote jack built in, this can be used with the 504 REMOTE UNIT, for patient with limited move ability.

This *Bed Head* unit is ideal for mounting into the patient bed head, because this unit can be canceled into the patient bed head.



## Call Button

The patient can press this button for nurse assistance. Once pressed the LED above this button will light-up. Multiple press of this button will not effect the system operation. And once triggered it's has to be reset again before operation. This "CALL" button is a very important button and the GAMA 500 Series Nurse Call System uses **normally closed type** button. In any case if this button fails the system will trigger automatically.

## Emergency Button

This button is used by the nurse to call for assistance in case there is a situation with the patient he/she attending.

Once this button is pressed the following will take effect.

- Main station audio alarm will be different to indicate emergency condition.
- The bed indication light at main station will flash.
- The Door Indicator will flash rapidly to indicate emergency condition.
- The Mute function will be disabled.
- The Call button will be disabled.

Only by pressing the related "RESET" button this emergency condition can be cancelled.

### Reset Button

As the name implies, this button is used to reset all the alarm conditions. The nurse who is attending the patient (who pressed the "CALL" button earlier), should press this button when he/she reached the patient bed.

This button can also be used as bed unit "TEST" button. When pressed in no-alarm condition all the LED's will light-up.

### Remote

This is where the hand-held remote jack is mounted. This remote jack is used as external "CALL" button for patient who has less mobility. Upon inserting the remote unit jack, the system will trigger the "CALL" button, indicating that the unit is connected. The nurse then should press the "RESET" button to reset the unit. After this the patient can press the remote "CALL" button to call for assistance.

### Technical Specification's

#### Power Source

Not needed, provided from the **GAMA 500 Series** main station

#### Dimension

Length : 145 mm  
Width : 85 mm  
Height : 12 mm (facial only)

#### Mounting

Can be mounted on standard Bed Head assembly, for non-bed head assembly please use standard electrical switch mounting base.